



Fitting Guide

Our frame design is different from what your opticians have fit in the past. Many opticians fit an oversized lens to compensate for the temple width that most of our patients need. We have solved this problem by making adjustments to our front end piece – we added up to 10mm on each end of the eye wire. This will mean less **decentration and a better looking lens**. For example if your patient fits a 40 eye size in a typical frame, they may fit our 38 eye size.

Our temple diameter on most models is the same throughout the length of the temple and can be cut down to size if needed. We also have spare temple tips to send upon request. Models #11, 12, 13 and 15 have flexible TR90 temples which cannot be cut down to size or extended, however we do have different lengths available upon request (110mm, 120mm, and higher).

Typical frames have the bridge aligned at the top of the frame. Since most individuals with Down Syndrome have a low flat bridge, standard frames will slide down their nose. **Erin's World Frames have a lowered bridge and temple placement** for balance and support.

Infants and toddlers with Down Syndrome usually experience sensory issues and have smaller close set ears; cable temples are usually a bother to them. With the balance and proper temple placement of Erin's World frames, cable temples are not needed. When needed, we highly suggest your patients try Speccles (www.speccles.com) as we do not have cable temples available.

Please keep in mind we do have spare parts available upon request: Screws, nose pads, and temple tips. These are not covered under warranty.