



# Fitting Guide

*Our frame design is different than what your opticians have fit in the past. Many Opticians are used to fitting an oversize lens to compensate for the width that most of our patients need. With the adjustments we made to our front end piece, by adding 10mm on each end of the eye wire, we have taken care of this issue. This will mean less decentration and a better looking lens. For Example if your patient fits a 40 eye size in a typical frame, they may fit our 38 eye size.*

*Our temple diameter is the same throughout the length of the temple and the temple can be cut down to size if needed. We also have spare temple tips to send if needed. Model #11-13 have a flexible TR90 temples. These cannot be cut down to size or extend. However we do have different lengths available upon request (110mm, 120mm, and higher).*

*Typical frames have the bridge aligned at the top of the frame. Since most individuals with Down syndrome have a low flat bridge, these frames will slide down to the bottom of their nose. Notice Erin's World Frames have a lowered bridge and temple placement for balance and support.*

*With balance in place and proper temple placement cable temples are not needed. Infant and toddlers with Down syndrome usually experience sensory issues and have smaller close set ears where cable temples are usually more of a bother to them then a help. We do highly suggest your patients try these straps [www.speccles.com](http://www.speccles.com) (we do not have cable temples available).*

*Please keep in mind we do have spare parts available upon request: Screws, Nose pads, and Temple tips, these are not warranty issues.*